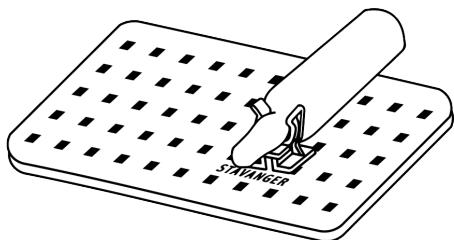
STAVANGER

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Please read these instructions carefully and completely before use and observe the care instructions.

Medications and other materials are clearly arranged in the STAVANGER® ampoule cabinet. The retaining clips developed by Fleischhacker secure sensitive glass ampoules and guarantee quick removal in case of emergency. The ampoule boxes are available in various sizes with different equipment. You may reorder all the individual parts.

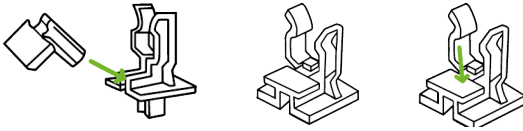
Initial commissioning and filling

The system module is delivered with pre-installed system plates. However, it can be extended using a center wall adapter (partly included in the scope of delivery) and an additional system plate. Four binding screws are required for mounting.


There are four different mounting options for organizing materials:

- **Retaining clip**


The reusable retaining clip is used to fix all common glass ampoules on system plates from 1 ml to 10 ml. It can be mounted either horizontally or vertically on the system plates. A locking pin must be locked in the retaining clip to secure it on the system plate (see illustration).

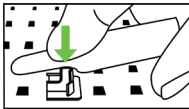
To do this, insert the locking pin into the aperture on the retaining clip and press it in until it locks. This pushes apart the retaining jaws in the base of the retaining clip.

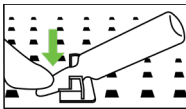Filling the ampoule by applying light pressure to the base of the neck

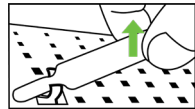Releasing the ampoule by applying light pressure to the tip of the ampoule

Removing the ampoule by lifting the body

The clamping clip on the retaining clip allows the glass ampoule to be held securely on the neck attachment without stressing the predetermined breaking point of the ampoule and at the same time allows a direct view of the label at all times. Changing glass ampoules of different sizes in a retaining clip should be avoided. Due to the balance between good elasticity and yet optimal dimensional stability, it is common for white discoloration to occur at the stress points. The retaining clip should be checked regularly and replaced if necessary.

- **Single Flex rubber bands**

The red Single Flex rubber bands are available in lengths of 35, 55, and 100 mm. The two T-splint ends are passed through a hole in the system plate and fixed in place. Please replace the bands in case of contamination.

- **Loop Flex rubber bands**

The LoopFlex is a 25-mm-wide, gray rubber band in various lengths. They are fixed to the system plate at the ends with a binding screw. The bars should be checked regularly and replaced if necessary.

- **Transparent pockets**

The transparent pockets, also available in different sizes, are designed for use on the center wall system plate. A firm connection to the system plate is achieved by using (2-4) binding screws.

The distance clip – a spacer – additionally protects the ampoules against glass breakage. Securing with the locking pin is recommended. If a STAVANGER® ampoule box is used with a third system plate, the spacer clip is necessary to keep the opposing glass ampoules at a distance. The positioning on the system plate can be individually adapted to the respective arrangement of ampoules. The additional use of foil pockets for damping is recommended. The distance clip should be checked regularly and replaced if necessary.

The labeling clip, together with the sticker set, is used for simplified filling of the ampoule tray and reduces placement errors. The special coating enables easy wipe-and-dip disinfection. The sticker set consists of 245 stickers with 71 of the most common active ingredients, including partly multiple copies and 5x colored stickers each in green, yellow, red, blue, and white for individual labeling.

Cleaning and Disinfection

The system module is made of robust truck tarpaulin and can be disinfected. With the exception of SingleFlex and LoopFlex, all components can be disinfected in case of heavy soiling or biological contamination by using medical surface disinfectants according to the specifications of the respective manufacturers. After each use and cleaning/disinfection, ensure that the product is completely dry.

Disposal

The product must be disposed in the accordance with local regulations.

Maintenance and Service

The end of the product's life is usually determined by wear and damage from use. Damaged parts must be replaced! For possible repairs, please contact your specialist dealer.

All parts of the ampoule box can be reordered. Please see the following page for article numbers.

Description

System plate XL Connect

System plate MM Connect

System plate XM Connect

System plate XL for center wall mounting

System plate MM for center wall mounting

Binding screw (20 pieces)

Retaining clip (20 pieces)

Locking pin (20 pieces)

Labeling clip (20 pieces)

Spacer clip (20 pieces)

Sticker set (active substance), 4 A4 pages

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Fleischhacker GmbH & Co. KG