



WESTMED

## DATA SHEET WESTMED® Single-hand tourniquet



### Order number

5810042









### Description

The strongly elastic WESTMED® one-hand vein tourniquet is very easy to use thanks to the multi-functional clip and guarantees quick application to the upper arm. It can be tightened and opened with just one hand.

Technical information	
Reference number	HS-403
Intended use	The tourniquet is used to staunch blood for intravenous puncturing of vessels for the purpose of taking blood or inserting an intravenous access. In addition, the tourniquet can also be used in an emergency to stop acute bleeding. This product may only be used by authorized specialist personnel!
Cleaning and disinfection	Please follow your internal hygiene rules to reduce the risk of cross-contamination. After application to a patient's skin, the tourniquet should be cleaned and disinfected with a 70% alcohol solution (e.g. with cloths soaked in it). Observe the exposure times of the product used.
Storage and transport conditions	The tourniquet must be stored and transported protected from dust, moisture and contamination.
Packaging unit	1 tourniquet per pack 300 packs per carton 24 cartons pro pallet
Lifetime	The end of the product's service life is normally determined by wear and damage caused by use. Damaged parts must be replaced!







General information	
Safety instructions	<ul style="list-style-type: none"><li>• The tourniquet must be kept out of the reach of persons who are not medically qualified.</li><li>• Check the complete tourniquet before use to rule out a defect. The product must not be used near or directly on the neck.</li><li>• Such use is associated with a fatal risk of strangulation and must be avoided at all costs.</li><li>• Do not position the closure directly on a pacemaker and never use it in an MRI.</li><li>• Before first use and after each use, cleaning should be carried out to prevent the colonization or transmission of germs. In the event of contamination with secretions, other body fluids or toxic substances, please follow the internal hygiene regulations and, if in doubt, dispose of as hazardous waste.</li><li>• Automatic cleaning or disinfection is not possible. Please follow the cleaning and disinfection instructions.</li><li>• Immediately after carrying out medical treatment, please open the cap as quickly as possible by pressing the multifunctional clip. Avoid uncontrolled popping when opening.</li><li>• Dispose of the tourniquet if there is visible wear on the band/closure.</li></ul>

Regulatory information	
Classification	Class I medical device According to Annex VIII, Regulation (EU) 2017/745
<b>Packaging symbols</b>	
	Medical device
	Reference number/article number
	Manufacturer
	Date of production
	Distributor
	Importer
	Authorized representative Europe
	Observe the operating instructions



WESTMED

DATA SHEET WESTMED® Single-hand tourniquet

	The CE marking is short for „Conformité Européenne“, which means „European Conformity“. It symbolizes the conformity of the product with the applicable requirements that the European Community places on the manufacturer.
	For marking a type BF applied part that complies with the IEC 60601-1 standard.
	Keep dry
	Protect against heat



HONSUN (NANTONG) Co., Ltd  
No. 8 Tongxing Road Economic & Technological Development Area,  
226009 Nantong City, Jiangsu Province, P. R. China



Shanghai International Holdin Corp. GmbH (Europe)  
Eiffestraße 80, Hamburg, Germany