



WESTMED

## DATA SHEET WESTMED® Standard blood pressure monitor



Order number

5810002

### Description

Our WESTMED® standard blood pressure monitor is very robust and an ideal alternative to the otherwise high-priced branded devices due to the price/performance ratio. The blood pressure values can be read easily and quickly thanks to the simple scale.




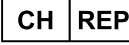







The 2-tube blood pressure monitor is supplied with a Velcro cuff for adults. As a suitable accessory, you can also get the WESTMED® Velcro cuffs for infants, children and adults with big arms.

Technical information	
Reference number	HS-201C2
Diameter scale	60 mm
Measuring range	0 - 300 mmHg
Measuring resolution	2 mmHg
Accuracy pressure display	± 3 mmHg (0.5 kPa)
Pressure reduction rate	2-3 seconds
Tubing system	2-tube-system
Sterilization	Non-sterilizable
Storage conditions	-20°C to 70°C at a relative humidity of 85% (non-condensing)
Transport conditions	Protected against light and water
Durability	10,000 measurements or 5 years from production date
Packaging unit	1 piece per package 30 packages per carton 16 cartons per pallet



General information	
Safety instructions	<ul style="list-style-type: none"><li>• Do not use if the blood pressure monitor is damaged or dirty.</li><li>• Never expose the cuffs to intensive solar radiation</li><li>• Never touch the cuff fabric or parts with a sharp instrument, since this could cause damage</li><li>• Always deflate cuff completely before storage.</li><li>• Do not dismantle manometer under any circumstance.</li></ul>
Cleaning and disinfection	Remove the bladder first and wipe the velcro, bladder and tubes with a damp cloth. The cuff can be washed with soap and cold water like all the other cuffs, but you must rinse the cuffs with clear water afterwards and let them air dry. Wipe off the manometer and bulb with a damp cloth. Sterilization is not necessary, since the parts do not come into direct contact with the patient's body.

Regulatory information	
Classification	Class I(m) medical device According to Annex VIII, Regulation (EU) 2017/745

Packaging symbols	
	Medical device
	Reference number/article number
	Authorized representative Europe
	Authorized representative Switzerland
	Distributor
	Importer
	Manufacturer
	Date of production
	Observe the operating instructions
	Keep dry
	Protect against heat



WESTMED

DATA SHEET WESTMED® Standard blood pressure monitor



HONSUN (NANTONG) Co.,  
Ltd No. 8 Tongxing Road Economic & Technological  
Development Area, 226009 Nantong City, Jiangsu Province, P. R. China



Shanghai International Holdin Corp. GmbH (Europe)  
Eiffestraße 80, Hamburg, Germany



MedNet SWISS GmbH  
Platz 4, 6039 Root D4, Switzerland